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In office single session treatment

GB BlancOne® TOUCH is an in-offi ce treatment that in just 30 minutes will immediately and effectively whiten teeth without the use of protective dams and without any dentin sensitivity.

USE INSTRUCTIONS

A) Preparing the gel

1. Remove the caps of the two syringes (whitening gel and activating gel) and connect them together using the luer-lock connector.
2. Gently push the piston of one syringe to completely extrude the gel into the other syringe.
3. Now push the piston of the syringe with the gel, transferring it into the empty syringe.
4. Repeat the operation described in the previous step until the two gels (whitening and activator) are completely mixed, creating a single homogeneous gel.
5. Remove the connector from the syringe with the gel and attach the applicator tip.

B) Preparing the patient

1. The patient should be fi tted with a BlancOne® gag with tongue depressor and then insert cylindrical tampons above and below the lips to isolate them from the area to be treated.
2. Remove the salivary fi lm with a brush; if necessary, use a mix of water and pumice stone. DO NOT use polishing pastes or products containing fl uorine. Rinse and dry.
3. Measure the patient's initial shade of white using the shade guide.

C) Whitening treatment

1. Gently push the piston of the syringe containing the whitening gel to apply the gingival gel on the surface of all teeth to be bleached.
2. Cure the whitening gel with a blue light.

Once activated, the gel will be swollen and full of oxygen (bubbles) as shown in the fi gure.

3. Remove the excess whitening gel using a cylindrical cotton roll and suction away the gel residues, rinsing with the air/water syringe.
WARNING: The activated gel will be rather dense.
If the suction spout becomes clogged, suction off with copious amounts of water.
4. The BlancOne® TOUCH treatment consists of 3 whitening cycles in a single session. Repeat the whitening cycle (C 1-3) 2 more times.
A fourth cycle can be added to further stabilize the result.

D) End of treatment

1. Remove any gingival barrier. Use the air/water syringe to rinse the surface of the teeth, suctioning off any gel residues.
NOTE: the gingival edge might be slightly red in some points but this is a temporary effect that will disappear in about one hour.

2. Use the shade guide to measure the fi nal shade of white of the patient's teeth.
3. Remove the gag and rinse the patient's mouth.

Photo activation of the gel

BlancOne® whitening gels require a suitable light source for the photochemical activation.

To guarantee the ideal whitening result, we recommend using BlancOne® ARCUS whitening lamp, which, being equipped with 10 powerful 5W led lights, guarantees the full activation of the gel simultaneously on both arches, including the sides, within the short time expected.
The head of BlancOne® ARCUS lamp should be positioned perpendicularly to the teeth at a distance of 5-10 cm from the mouth to guarantee a good ventilation.

Using other whitening lamps might reduce the effectiveness of the treatment, require a longer time or, in some cases, completely fail to activate the whitening gel.

RECOMMENDED ACTIVATION TIME			
Equipment	Power	Type of activation	Time
BlancOne® ARCUS Lamp	250 mW/cm²	simultaneous on both arches	8 minuti

EUROPEAN UNION WARNINGS

- 5,6% of H₂O₂ is released.
- Not to be used on anyone under 18 years of age.
- To be sold only to dental practitioners.
- To be used only by dental practitioners or under their direct supervision if an equivalent level of safety is ensured.

PRECAUTIONS

- A dental examination is mandatory prior to bleaching.
- Bleaching will not lighten porcelain crowns and dental restorations.
- For at least 6 hours after the bleaching procedure do not use tobacco products and do not drink coffee or red wine.
- Not intended for use by pregnant women. Consult a doctor if you are breast feeding.
- Do not swallow the gel. Avoid contact with tissues other than teeth.
- Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
- Gum tissue irritation may result if used in individuals with periodontal disease.
- Keep teeth bleaching products away from children.
- Avoid contact with skin, mucous membranes and eyes. In case of contact with EYES: rinse eyes carefully with water for several minutes and request medical advice/attention. ON SKIN/ MUCOUS MEMBRANES: wash with copious amounts of water. If irritation occurs request medical advice/ attention.
- If swallowed in small quantities the product is harmless, otherwise consult a doctor.
- Some users may experience sensitivity or gingival irritation.
While these are mild and transient in most cases, sometimes treatment by a dentist may be required.

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